



Instructions for the assembly and safe use of
The Buckingham foldeasy2™

Contents

1 X Buckingham foldeasy2™


Code: FTFS

Max user weight 127kg (20 stone)

Manufactured in China for SmitCare

Important Notes

A risk assessment should be carried out as to the suitability of this product particularly with new users.

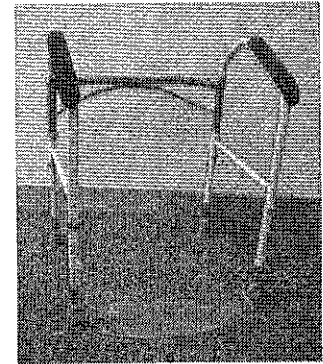

1. The frame is supplied folded; to open the frame rotate each side in turn until it resembles the image above and the sides are parallel to each other. When the sides are parallel, the frame will be locked in this position by two sprung buttons on the grey frame section (Fig 1).

Fig 1

2. The height of the frame may be adjusted as desired, or as directed by a Health Professional. To adjust the height of the frame you need to adjust one leg at a time by removing the C-clip from the frame, sliding the leg up or down as required, then re-inserting the C-clip to retain the position. Ensure the C-clip is fully through the frame for security (Fig 2).
Note: When changing the height of the legs, it is important that all the legs are at the same height setting and that the frame is stable before use.

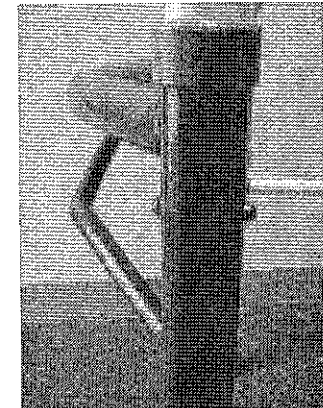


Fig 2

3. Once the frame is locked open and adjusted to the correct height, the footplate should be lowered and clipped on to the securing pin of the opposite rubber ferrule (Fig 3)

Note: The foldeasy2 should only be used once it is fully unfolded with the sides locked in the open position and the footplate secured. When using your foldeasy2 try to push down equally on both armrests when positioning yourself over the toilet.

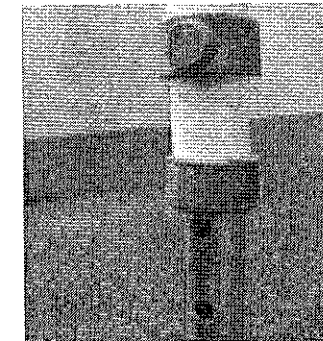


Fig 3

4. To close the foldeasy2 after use, reverse the above procedure. Raise the footplate until it clips to the upright tube, then fold in the two frame halves by depressing sprung buttons shown in Fig 1 and rotating inwards.

Cleaning: Clean with a solution of mild detergent and household disinfectant in warm water applied with a damp sponge. Dry thoroughly. Do not clean with harsh abrasives.

Warranty period one year from date of purchase.

Questions / comments? Please contact our customer services department
on Lo-call : **0845 1303 747**

