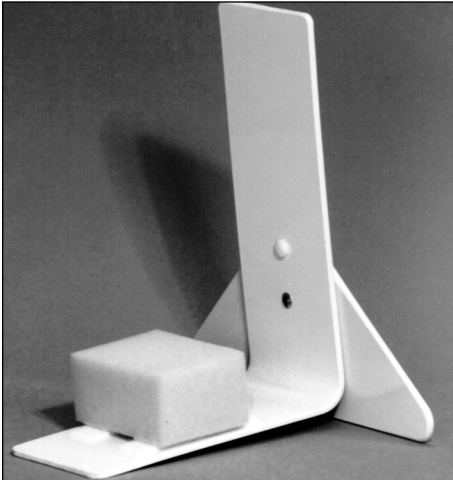
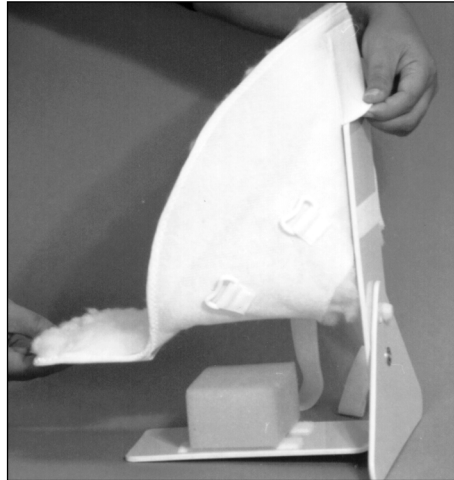
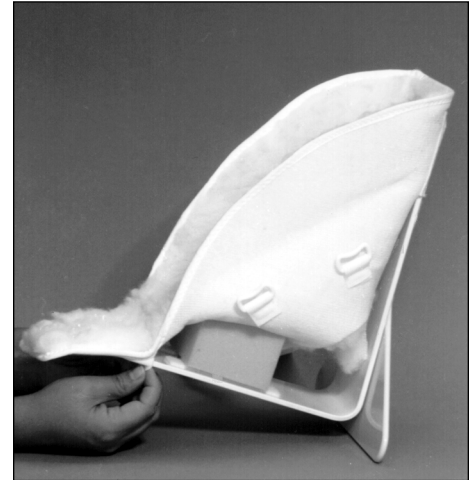
**A**

**B**

**C**

#### INDICATIONS

- To help prevent foot drop, tendon shortening, hip rotation, and plantar flexion contractures of the ankle

#### CONTRAINDICATIONS

- Not for severe hypertonicity
- Not for fixed bony contractures in plantar flexion
- Not for use with fluctuating edema
- Not for use over open wounds and infected areas

#### PRECAUTIONS

- **This product is not to be used during ambulation or for bearing weight.**
- **This product is to be fitted initially by a healthcare professional who is familiar with the purpose for it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, discomfort, or skin discoloration or breakdown occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **This product is recommended for use by one patient only.**
- **This product should not be used without the foam block.**

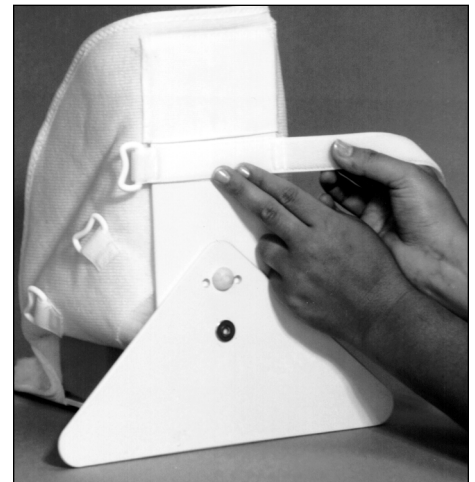
#### CARE AND CLEANING

Remove boot, foam block, and loose D-ring strap from base prior to cleaning.

- Base: clean with soap and water or a hard-surface disinfectant and wipe dry.
- Boot and loose strap: Machine wash in warm water and machine dry at low temperature. Be sure they are thoroughly dry before reapplication.
- Open-cell foam block: Surface wipe and allow to air dry thoroughly before reapplication.

#### INSTRUCTIONS FOR USE

1. Position triangular support so hole in foot portion of base lines up with one of the holes in triangular support. Holes provide adjustability of 20° to either side for internal or external rotation.
2. Remove thumb nut from slotted bolt. From side of base that will be against foot, insert bolt through holes in base and triangular support (A). Secure with thumb nut.
3. Attach hook on foam block to loop on inside of base (A).
4. Position base and triangular support with long edge of triangular support down.
5. Position boot with D-rings to side, rounded edge with elastic pocket up, and long flap with elastic pocket near foam block in base (B).
6. Put elastic pocket on back of top of boot over upper end of base (B). Hook on base should be just below upper elastic pocket.
7. Put elastic pocket that is on long flap of boot over other end of base (C).

**D**

8. With D-ring close to edge of base, attach loose strap to hook on base (D).
9. Open flaps of boot and place foot inside with ankle resting on foam block.
10. Close flaps over ankle and foot and fasten straps across foot. They can be straight across, or for added support two proximal straps can be crossed over ankle.

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*(Design copyright Smith & Nephew.)*
**Patterson Medical Ltd.**