	NEW CODE	OLD CODE
Uni-Frame Folding Shower Chair	09 107 5795	AA1614
Uni-Frame Folding Toilet Rail	09 107 9011	AA2084
Uni-Frame Folding Toilet Frame:	09 108 0290	AA2264
Uni-Frame Folding Commode:	09 108 0803	AA2334
Screw-Down Feet:	09 107 9003	AA2082

Instruction Sheet

Intended uses

- To help increase independence when using the shower or toilet, particularly by compensating for impaired movement.

Assembly and Use

Basic Frame

• Adjusting the height (Figure 1):

With the frame lying on the floor, remove the pin-clips **(A)**, which release the leg sleeves **(B)** and the legs **(C)**. Adjust to the desired height, align the holes, and replace the pin-clips. Be sure all of the legs are adjusted to the same height.

• Attaching the components (Figure 4, showing the Shower Chair):

The frame can be used with a variety of rails and seats. Attach the components as follows:

1. Clip the back of the rail or seat **(D)** onto the back bar of the frame **(G)** and push firmly into place.
2. Position the front pegs on the rail or seat into the holes **(F)**.
3. Position the back pad **(E)** in the centre of the back of the upper frame and push it firmly into place.

• Folding the frame (Figures 2 and 3):

To fold the frame, pull the front pegs out of the holes and swivel the rail or seat down. Then fold one set of legs across the front of the frame and the other set across the back.

• Using the optional screw-down feet:

If the frame is to be used in only one place, it can be attached permanently to the floor using the screw-down feet (09 107 9003). To use the screw-down feet, remove the leg sleeves with the slip-resistant feet and attach the screw-down feet in their place. **Attachment to the floor should be done by a qualified tradesman.**

Uni-Frame Folding Shower Chair (09 107 5795) - see Figures 4 and 5

- Attach the shower chair **(D)** to the frame as described in **Attaching the components**.

Uni-Frame Folding Toilet Rail (09 107 9011) - see Figures 6 and 7

- Attach the surround rail **(H)** to the frame as described in **Attaching the components**.

Uni-Frame Folding Toilet Frame (09 108 0290) - see Figures 8 and 9

- Attach the rail **(J)** to the frame as described in **Attaching the components**.
- Position the seat **(K)** in the centre of the frame and apply pressure to the seat to clip it securely into place.

Uni-Frame Folding Commode (09 108 0803) - see Figures 10 and 11

- Attach the rail **(L)** to the frame as described in **Attaching the components**.
- Position the commode pot **(P)** so it rests on the bars of the commode rail.
- Remove the commode pot lid **(N)**.
- Rest the commode seat **(O)** on the rail so the attachment pins fit inside the bars on the commode rail.
- The back panel **(M)** can be used as a backrest while the commode is being used and as a lid.

Care and Cleaning


All parts can be wiped using a brush or cloth and a solution of mild detergent or household disinfectant.

Cautions

- The Uni-Frame Folding Shower Chair (09 107 5795) is not to be used to support more than 120 kg / 264 lb / 18³/₄ st; the Uni-Frame Folding Toilet Frame (09 108 0290) is not to be used to support more than 127 kg / 279 lb / 20 st.
- Be sure the frame and accessories are securely in place before putting weight on them.

Uni-Frame Folding Shower Chair (09 107 5795)

- Use of a slip-resistant bath mat is strongly recommended.
- Cover the seat with a towel to help reduce slipperiness.

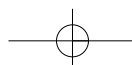N NEDERLANDS**Uni-Frame opklapbare douchestoel, Uni-Frame opklapbare toiletrail, Uni-Frame opklapbaar toiletframe, Uni-Frame opklapbare toilettafel en aanschroefbare voeten**

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