# Homecraft Curved Transfer Board

NEW CODE	OLD CODE
09 112 0989	AA8835

## Intended Uses

- To compensate for a disability by helping a person transfer from one sitting surface to another.

## Usage Restrictions

- Not for people who have an allergic reaction to PVC or who have a tendency to develop dermatitis.

## Instructions for Use

- With the slip-resistant pads down, position one end of the board under the user's body and the other end on the destination surface. Position the board so it supports the full body weight during transfer and the curve

facilitates the transfer.

- The smooth surface of the board is best suited for clothed transfers. For bare-skin transfers, a towel can be placed over the board for a smoother transfer.

## Care and Cleaning

The polypropylene board with PVC pads can be cleaned with a household disinfectant or mild detergent and water.

## Cautions

- It is recommended that the Curved Transfer Board not be used without previous training by a healthcare**

professional. For some people, attempting a transfer without assistance from a care giver is not recommended.

- The board is not to be used to support more than 152 kg / 336 lb / 24 st.
- Inspect the board prior to each use for signs of wear. Do not use the board if it appears weakened.
- Be sure the board is securely in place before putting weight on it.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

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