

## Intended Uses

Commodes allow for, or increase, independence when requiring the toilet, particularly by compensating for impaired movement or disability.

## Description

Attractive, sturdy commode chair that has drop down arms to allow side transfer. The height adjustable legs allow the seat to be positioned at different heights. For additional comfort, the armrests are padded. Supplied with a toilet seat and commode pan.

## Instructions for use

1. The commode is supplied assembled
2. Insert the commode pan at the front or rear of the commode
3. Simply press the release lever at the front of the armrest to drop the arms down
4. When seated, lift the armrest and lock into position

## Cleaning and care

Clean frame and seat with mild non-abrasive cleanser and water, then towel dry. Clean after each use with mild disinfectant, then towel dry.



### Cautions

- The commode is not to be used to support more than 136kg/300lbs/21¼st
- Make sure all push buttons on extension legs are aligned equally (same height adjustment) and all push-button pins are fully extruded to assure a locked, secure position
- When the armrests are located in the horizontal position, please ensure the armrests are locked into position before use
- Before use, make sure all screws are tight and all push-button pins are correctly positioned
- Under no circumstance should the commode be moved whilst occupied



Maximum  
user weight



These user instructions are available in large print, please contact Patterson Medical Customer Services for a copy.  
Please retain these instructions for future reference.

### Manufactured for Patterson Medical Ltd.

Nunn Brook Road, Huthwaite,  
Sutton-in-Ashfield, Nottinghamshire,  
NG17 2HU, UK.

UK: Tel: 08444 124 330 Fax: 08448 730 1000  
www.pattersonmedical.co.uk

International: Tel: +44 1623 448 706  
Fax: +44 1623 448 784

### Patterson Medical Products, Inc.

1000 Remington Boulevard, Suite 210,  
Bolingbrook, IL 60440-5071, USA.  
Tel: 1-800-323-5547 Telefax: 1-800-547-4333

### Patterson Medical Australia

PO Box 1006, North Ryde, NSW Australia 2113.  
Tel: 1300 473 422 Fax: 1300 766 473

### Patterson Medical France

Kinetec SAS, Zone Industrielle de Montjoly,  
11 Avenue de l'Industrie - CS 40712  
08013 Charleville-Mézières Cedex, France.  
Tel: 03 24 52 61 15 Fax: 03 24 52 90 34

