

1. Kings Standard - Weight Built-Up Handle

Four handles are available. The Standard has a hollow cylindrical body; the Slim has an oval cross-section. This handle is also used as the core for the Vari-grip sleeves. The Contoured has a curved shape and finger contours. The Mushroom is a round hollow door knob type handle which is easy to grip for those with weak or painful joints.

Dimensions and weights:

- Standard: 110 x 25.5mm; 30g.
- Slim: 100 x 14-19mm; 29g.
- Contoured: 120 x 22 - 30mm; 85g.
- Customised as req'd 65g.
- Mushroom length: 40mm (1 3/5");
Diam. 65mm (2 1/2"); 52g.

AA5536 Standard
AA5543 Slim

AA5535 Contoured
AA5544 Mushroom

2. Kings Heavyweight Handle

This version of the standard built-up handle AA5536 has the hollow core filled with a non-toxic metal bar, to create a heavier handle for those with tremor etc.

Dimensions and weights:

- Heavy Handle 110 x 25.5mm; 156g.
- Final utensil weight approx. 176g.

AA5539 Heavy handle

3. Kings Vari-Grip Built-Up Sleeves

Contoured PVC sleeves that fit on to the slim handle AA5543. This replaces the Selecta-grip range. The oval sleeve is oval in shape in both dimensions; the small and large contoured sleeves are contoured with finger grips.

Dimensions and weights:

- Oval sleeve 105 x 23 - 30mm; 24g.
- Small sleeve 105 x 22mm; 19g.
- Large 105 x 29 - 35mm; 38g.

AA5345 Oval sleeve
AA5546 Small sleeve

AA5547 Large sleeve

4. Kings Lightweight Foam Handles

These are made from grey sculptured closed cell foam for those unable to cope with the weight of standard built-up handles. They are waterproof but washable. Both the small and large diameter version have contoured finger grips.

Dimensions and weights:

- Small 118 x 20mm (4 3/5 x 4/5"); 10g.
- Large 118 x 28mm (4 3/5 x 1 1/10"); 12g.
- Final utensil weight approx 32g.

AA5538 Small
AA5537 Large





5. Kings Double Ended Handles with Strap

These are versions of the standard built-up and lightweight foam handles with utensil apertures in both ends. Supplied with a transparent adjustable strap and a fixing spigot that fits in the second hole A second utensil can be used e.g. the rocker knife AA5529 or angled knife AA5530 with or without the strap, to create versatile one handed cutlery

Dimensions and weights:

- Strap measurements 210x20mm (8x 3/4")
- Standard Double 25mm: 30g
- Lightweight Double 28mm: 12g

AA5540 Standard Double
AA5541 Lightweight Double

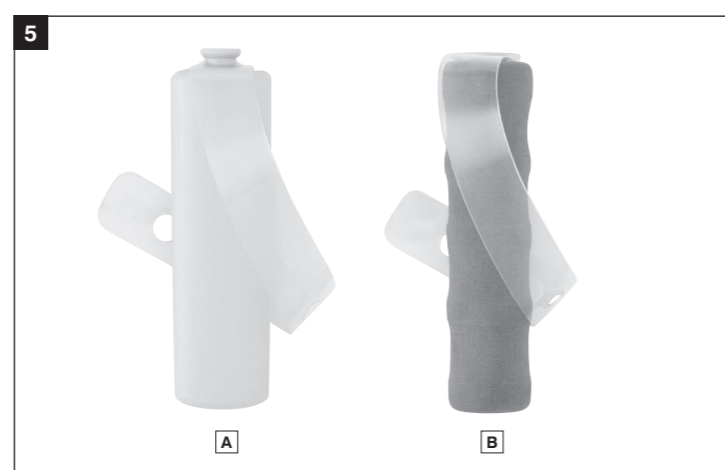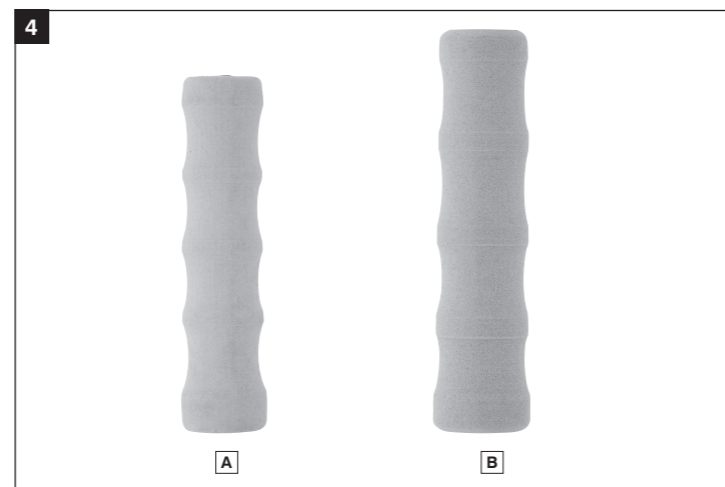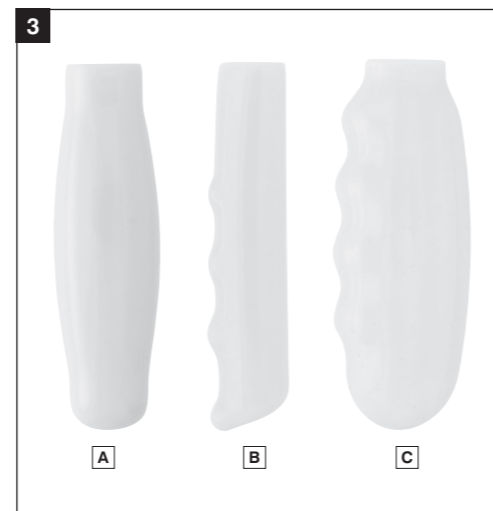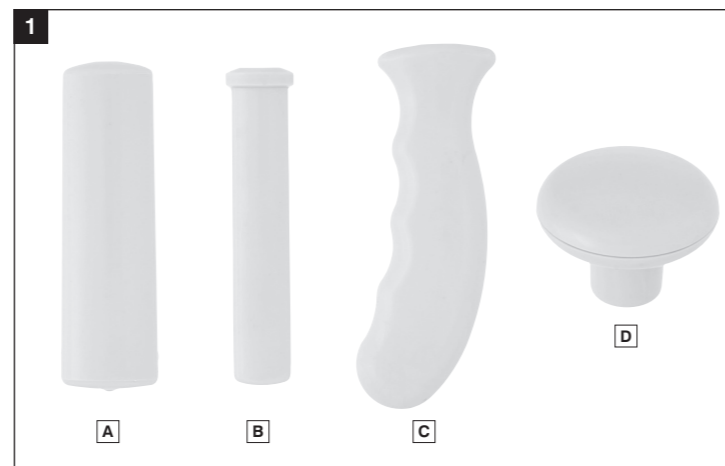

Intended uses


This attractive and comprehensive range allows over 400 individual cutlery items to be made from 32 modular items.

It caters for virtually all needs, economically, visually, aesthetically and functionally.

The range covers: standard, angled and specialised utensils; built-up, slim, contoured, lightweight and heavyweight handles; and a unique double-ended feature to create special utensils for one-handed eating or to provide an inconspicuous hand strap.

The 18 utensils, angled extension, 13 handles and hand strap are completely integrated, and designed to a high standard of quality.

All items are hygienic, non-corroding and dishwasher-safe.

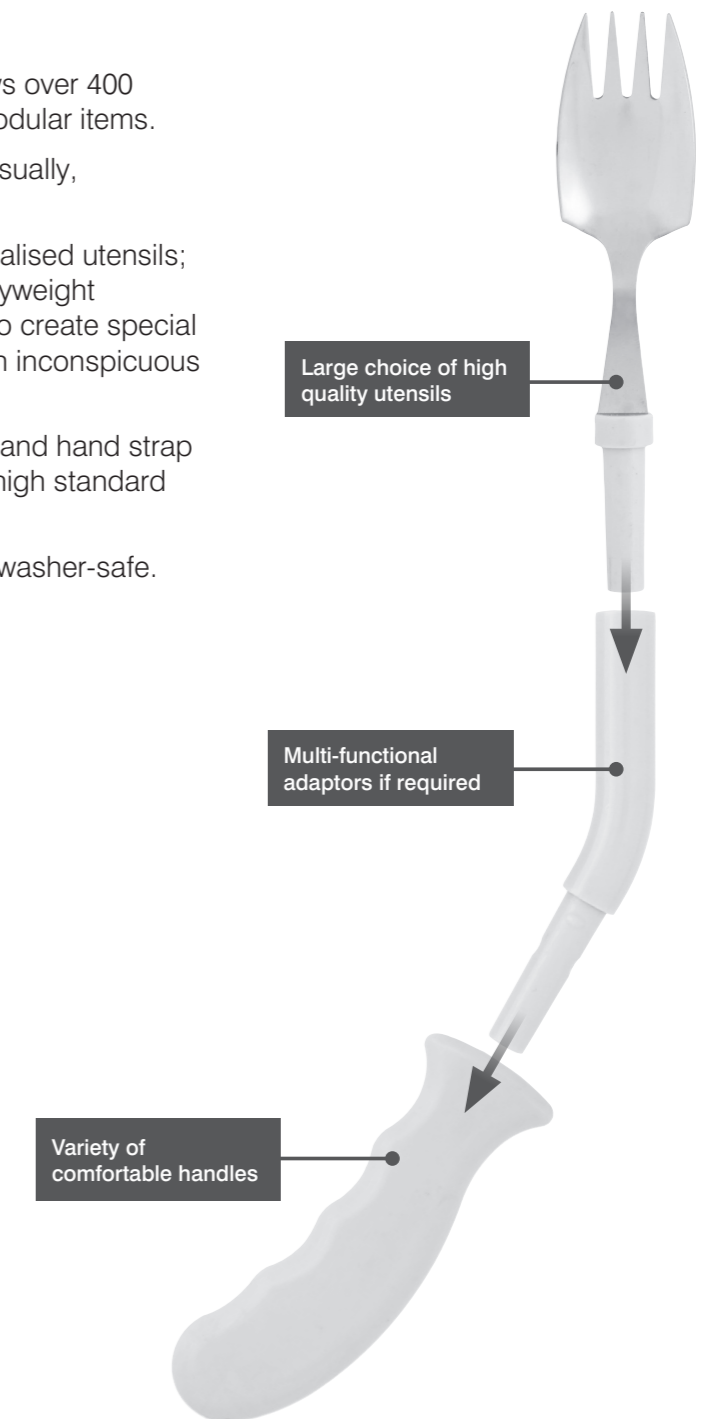

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for
Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health Australia

Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

1. Kings Standard Cutlery Utensils

These standard utensils are mounted in plastic spigots that push fit tightly into the chosen handle, but allow the blade to be angled as desired.

	Description	Dimension	Weight
AA5520	A Knife	112mm	17g
AA5521	B Fork	104mm	21g
AA5522	C Std. Spoon	97mm	23g
AA5523	D Junior Spoon	91mm	21g
AA5524	E Teaspoon	84mm	17g
AA5526	F Soup Spoon	80mm	21g

2. Kings Angled Cutlery Utensils

A range similar to 1 above with angled blades for those with limited wrist or arm movement.

The knife is at 90 deg. and can also be used as the second utensil in the double-ended handles. Forks and spoons have left and right handed versions at 40 deg. angles. The angled adaptor adds extra length and a 30 deg. angle to any utensil.

- Knife is 52mm with 80mm offset at 90 deg. angle.
- Spoons have a 40 deg. angle.
- Adaptor has a 30 deg. angle.

	Description	Dimension	Weight
AA5530	A Knife	132mm	20g
AA5531L	B Forks	82mm	21g
AA5531R	C Forks	82mm	21g
AA5532L	D Spoons	82mm	23g
AA5532R	E Spoons	82mm	23g
AA5534	F Adaptor Pk x 3	60mm	10g

3. Kings Specialised Cutlery Utensils



The Splayed is a combination knife, fork and spoon and the Nelson knife is a combined knife and fork for one-handed eating. The small Rocker knife is a short angled knife for cutting food with minimal rocking movement, and can be used in the double-ended cutlery handles to make one utensil more versatile. The larger Rocker knife has a broad semi-circular blade which is ideal for preparing larger food without the cocking of the wrist.

	Description	Dimension	Weight
AA5527	A Splayed	82mm	22g
AA5528	B Nelson knife	110mm	25g
AA5529	C Small Rocker	35mm with 60mm width	19g
AA5533	D Large Rocker	Max. width of blade 121mm	45g

4. Kings Soft Coated Spoons

These spoons have been dipped in soft food quality white PVC. This makes the spoons ideal for users with bite reflex problem when eating. Due to the type of use these items are subjected to, they are guaranteed for 3 months.

	Description	Dimension	Weight
AA5518	A Spoon	100mm (4")	32g
AA5519	B Teaspoon	80mm (3 1/5")	25g

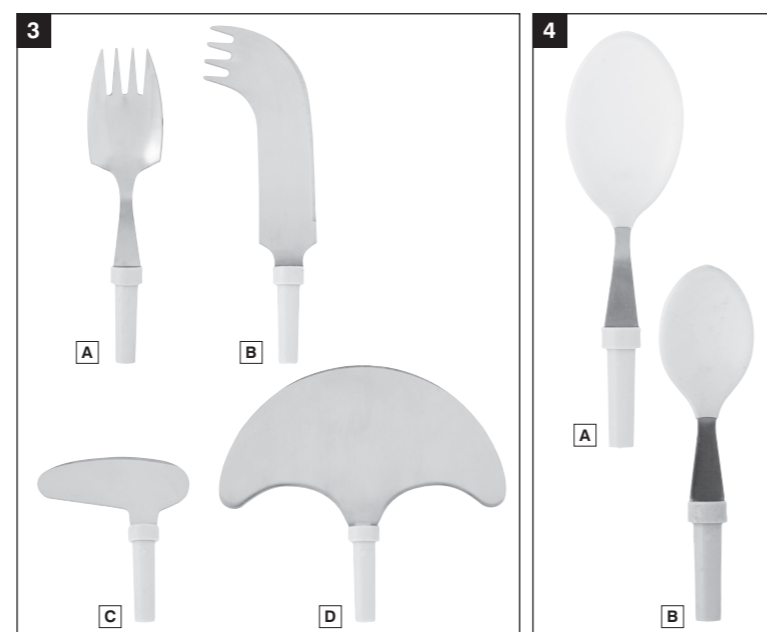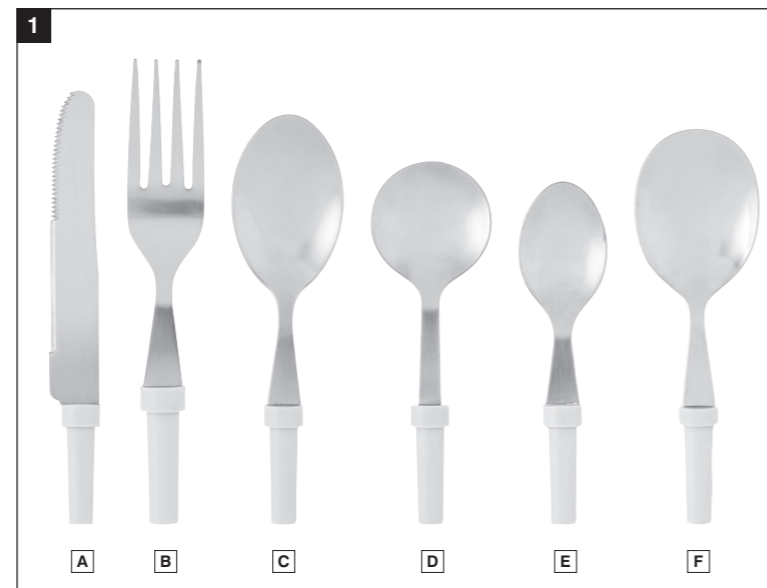

1. Assembly Of Kings Range

All parts of the Kings Cutlery are fitted together simply, creating items suited to the individual.

- To assemble a cutlery utensil and a handle, you simply push the utensil spigot into the hole in the handle.
- If you require a double ended utensil, a handle with a hole in both ends must be purchased, the utensil(s) can be rotated to the position required.
- If an additional angled adaptor is needed, the adaptor is pushed onto the utensil spigot and the adaptor then pushed into the hole in the handle.
- When using the strap, it is placed between the handle and cutlery spigot or end piece.
- If the handle needs to be built up further, sleeves can be added, these must be fitted over the AA5543 slim handle, and then the handle attached to the utensil in the manner described above.

2. Kings Assessment Kit - 33 Pieces

This kit contains one of each utensil, handle and sleeve, plus the Kings Button Hook. Ideal for assessing individual patient needs. Supplied in an attractive case with a specific place for each item.

AA5548

