

Intended Uses

Commodes allow for, or increase, independence when requiring the toilet, particularly by compensating for impaired movement or disability.

Description

Attractive, sturdy commode chair that has drop down arms to allow side transfer. The height adjustable legs allow the seat to be positioned at different heights. For additional comfort, the armrests are padded. Supplied with a toilet seat and commode pan.

Instructions for use

1. The commode is supplied assembled
2. Insert the commode pan at the front or rear of the commode
3. Simply press the release lever at the front of the armrest to drop the arms down
4. When seated, lift the armrest and lock into position

Cleaning and care

Clean frame and seat with mild non-abrasive cleanser and water, then towel dry. Clean after each use with mild disinfectant, then towel dry.



Cautions

- The commode is not to be used to support more than 136kg/300lbs/21¼st
- Make sure all push buttons on extension legs are aligned equally (same height adjustment) and all push-button pins are fully extruded to assure a locked, secure position
- When the armrests are located in the horizontal position, please ensure the armrests are locked into position before use
- Before use, make sure all screws are tight and all push-button pins are correctly positioned
- Under no circumstance should the commode be moved whilst occupied



Maximum
user weight



**These user instructions are available in large print, please contact Performance Health Customer Services for a copy.
Please retain these instructions for future reference.**

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health ANZ

Unit 3, 3 Basalt Road,
Pembury NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

**PERFORMANCE
HEALTH**