
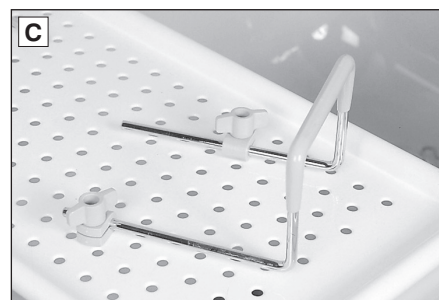
NEW CODE	OLD CODE	US CODE
09 107 2479	AA1080	08 130 5010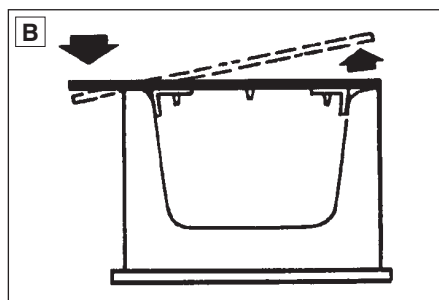

Intended uses

- To compensate for a disability by providing a sitting surface for a person taking a bath or shower.

Instructions for Installation

- Loosen the wing nuts that fasten the brackets to the bottom of the board. Slide the brackets as far toward the center of the board as possible.
- Place the board on the bathtub, being sure that the board rests securely on both sides of the tub without excessive overhang (**A and B**).
- Move the brackets toward the sides of the bathtub until they are snug against the sides. The brackets can be placed at an angle to accommodate curvature of the tub (**C**).

- Tighten the wing nuts securely.
- Repeat the fitting procedure each time the bath board is placed on the bathtub.
- The bath board can be used with a bath seat that enables the user to sit lower in the bathtub.

Care and Cleaning

The board can be cleaned with a household disinfectant or mild detergent and water.

Cautions

- The bath board is not to be used to support more than 190kg / 420lb / 30st.

- Both ends of the bath board must be supported securely by the sides of the bathtub.
- Avoid excessive overhang, which may result in tipping of the bath board (**B**).
- Be sure the bath board is securely in place and the brackets are snug against the sides of the tub before putting weight on the board.
- Use of a slip-resistant bath mat is strongly recommended.
- Cover the bath board with a towel to help reduce slipperiness.

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Manufactured for Performance Health International Ltd.
Performance Health Supply, Inc.