E Barra de apoyo 'Devon'

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These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

28100 Torch Parkway Suite 700, Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health ANZ

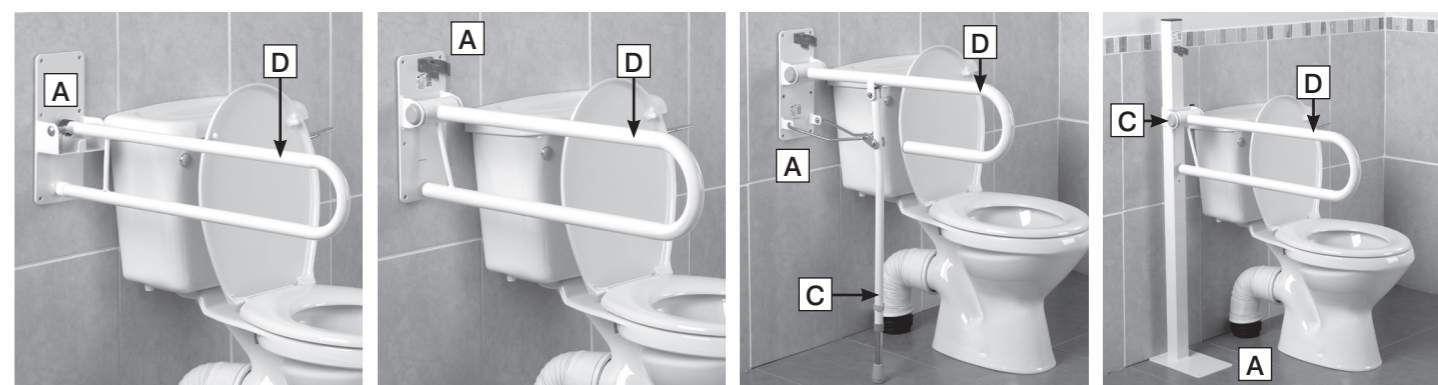
Unit 3, 3 Basalt Road, Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

Zone Industrielle de Montjoly, 11 Avenue de l'Industrie - CS 40712 08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

PERFORMANCE HEALTH

homecraft Devon Support Rail

AA2000, AA2002

Maximum load weight **67 kg** **10 1/2 st**

AA2010, AA2012

Maximum load weight **84 kg** **13 1/4 st**

AA2018

Maximum load weight **67 kg** **10 1/2 st**

AA2024, AA2026

Maximum load weight **127 kg** **20 st**

Intended uses

- To compensate for a disability by helping support a person while standing up and sitting down

Instructions for installation

1. These rails should be installed by a person who has suitable qualifications and/or experience in installing this type of product, ensuring the fixtures are appropriate for the specific installation
2. All fixing holes in the wall plate or floor plate (labelled A) must be used. The rail must be fastened securely enough that it is able to support the intended user at point D. The intended user must not exceed the maximum load weight specified right
3. All of the rails can be folded up out of the way when not in use
4. It is recommended that the security of the fixings for the rail should be checked annually or more frequently, depending on usage and location of the rail. Refer back to point 2 for details

Care and Cleaning

The rails can be cleaned with mild detergent and water.

Cautions

- Ensure that the chosen rail is suitable for use by the intended user
- All rails should be installed by a person who has suitable qualifications and/or experience installing this type of product, who can verify that the rail is fitted correctly
- Before each use, ensure that the rail is secure before putting weight on it.
- On height-adjustable models, ensure the height-adjustment knob is securely tightened
- Ensure that the rail is dry before each use

Please carefully read the instructions before installation and use of the rail.

Product code	Maximum load weight	Height Adjust	Installation Instructions/Comments
AA2000	67kg	No	Must be permanently attached at the optimum height
AA2002	67kg	No	Must be permanently attached at the optimum height
AA2010	84kg	No	Must be permanently attached at the optimum height
AA2012	84kg	No	Must be permanently attached at the optimum height
AA2018	67kg	Yes	The leg length must be adjusted prior to installation, as follows: (1) Hold the wall plate against the wall and extend the rail to approximately a 45° angle with the wall. (2) Remove the pin clip (C) from the leg, determine the appropriate height, and replace the pin clip. (3) When determining the position of the wall plate for installation, be sure the tip of the leg is securely on the floor.
AA2024	127kg	Yes	Must be mounted to the floor. To adjust rail height, loosen the knob (B) and slide the rail to the desired height. Tighten the knob.
AA2026	127kg	Yes	Must be mounted to the floor. To adjust rail height, loosen the knob (B) and slide the rail to the desired height. Tighten the knob.

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