



Description

The Ringwood Wall to Floor Grab Rail is made from large 32mm (1¼") steel tubing for comfort and grip.

The rail has a white epoxy coating finish making it suitable for indoor or outdoor use. The assembled rail measures 760mm (30") from the wall to which it is attached and is 780mm (30¾") high.

The rail has a three hole circular fixing plate at each end for safe and secure attachment. For easy storage and transportation the rail comes in two pieces and is self-assembled using a bullet catch mechanism.

Intended Use

To compensate for a disability by helping to support a person with reduced balance and / or mobility when using the toilet.

Installation

The rail should be installed and the appropriate screws selected by a skilled tradesman. The two parts of the rail should be assembled

and checked to ensure they are secure prior to installation.

To assemble the rail, squeeze the butterfly catch mechanism on the vertical portion of the rail so that the horizontal portion can fit over it. The bullet catch should fit into the pre-cut holes. Prior to being attached to the intended wall to which it is being fitted, line up the grab rail and mark where the fixings are to be applied.

The rail should be fitted so that the joint between the two parts of the rail is positioned in the vertical section. All rails must be fixed to a strong secure wall whether used indoors or outside. The maximum user weight/load is dependent upon the correct choice of fixings for the mounting surface and it is for this reason we recommend installation by a qualified tradesman.

Cleaning and care

Wipe over with a damp cloth and a solution of household detergent or disinfectant.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health ANZ

Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

**PERFORMANCE
HEALTH®**