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homecraft Devon Elite Folding Support Rail

CODE	Product
09 116 5695	Devon Elite Folding Rail Standard
09 117 3145	Devon Elite Folding Rail With Folding Leg Standard

CODE	Product
09 117 3152	Devon Elite Folding Rail Adjustable Height Standard
09 117 3160	Devon Elite Folding Rail Floor Mounted Standard

Intended Uses

To compensate for a disability by helping support the user while standing up and sitting down. Instructions for installation and use

- These support rails should be installed by a qualified tradesman, who will choose the appropriate hardware for the specific installation.

- All fixing holes in the wall plate or floor plate must be used. The rail must be fastened securely, so it is able to support the total load applied on the rail.

- Model **09 116 5695** is not height adjustable, and they must be permanently attached at the optimum height for the user.
- Model **09 117 3145** include a height adjustable leg, which must be adjusted prior to installation, as follows:

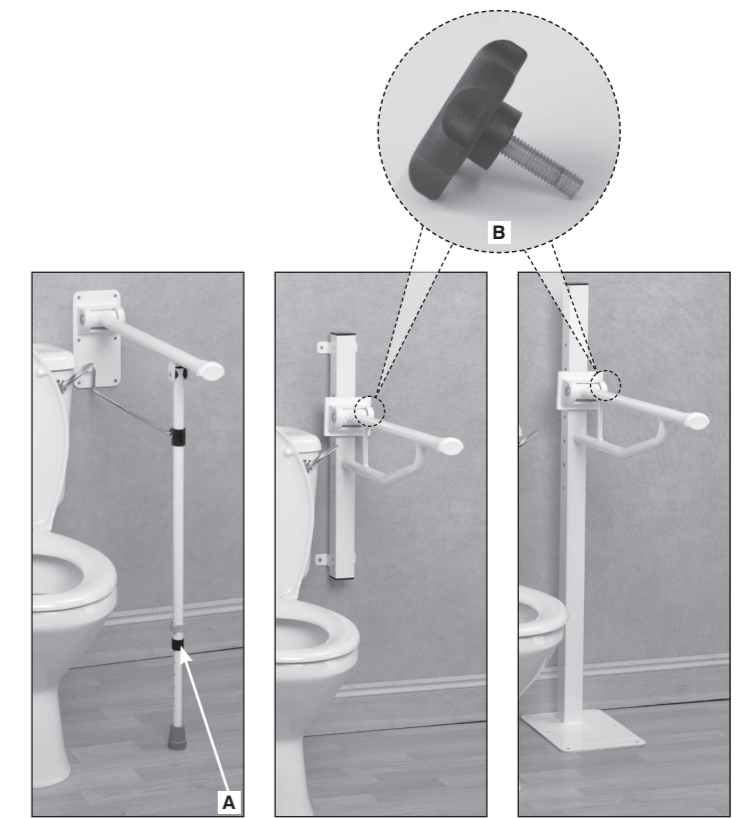
- Hold the wall plate against the wall and extend the rail to approximately a 90° angle with the wall.
- Remove the pin clip (A) from the leg, determine the appropriate height, and replace the pin clip (A).
- When determining the position of the wall plate for installation, be sure the ferrule of the leg is securely on the floor.

- Model **09 117 3152** is height adjustable, and should be attached with the optimum height approximately in the centre of the column. To adjust the rail height, loosen the knob (B) and carefully slide the rail up and down to the desired height, ensuring the peg is located in a hole. Tighten the knob (B). The black plastic plug will protect the paintwork, when the knob is tightened.

- Model **09 117 3160**, which is height adjustable, must be securely mounted to the floor. To adjust the rail height, loosen knob (B) and carefully slide the rail up and down to the desired height, ensuring the peg is located in a hole. Tighten the knob (B). The black plastic plug will protect the paintwork, when the knob is tightened.

- The rails can be lifted and secured in the upright position.

- Lift the rail from the locking mechanism and allow the rail to lower slowly to the level position. There is no need to keep hold of the rail while lowering.

Care and Cleaning

The rail can be cleaned with mild detergent and water. Must be dried after cleaning and before use.

Cautions

- We recommend that the rail be installed by a qualified tradesman, for this reason we do not supply fixings
- Be sure that the rail is securely in place before applying downward force
- The rail is not intended to take the full weight of the user
- The rail is designed to take a maximum load weight of 127 kg/ 275 lbs/ 20 stone

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for

Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

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