

days® Overbed Table

New Code	Old Code	Description
09 132 5653	751	Days Overbed Table without Castors
09 132 5745	751C	Days Overbed Table with Castors

Introduction

The Days model No. 751 and 751C are adjustable height overbed tables made from chromium plated and powder coated steel tubing and can also be used at a chair or wheelchair. The easy-clean laminated teak top has raised edges with open corners and can be tilted for reading.

The 751C model is fitted with castors which can be braked to avoid movement of the table when it is in the desired position.

⚠ CAUTION

The overbed tables are not designed to support heavy weight. Users must not lean on them or use them for support when getting up or sitting down.

Height Adjustment

The table height can be adjusted by slackening the two black hand knobs and adjusting the table top to the desired position then tightening the hand knobs. Before use make sure that both hand knobs are securely tightened.

Maintenance

The Days overbed tables are designed for minimum maintenance.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Specification

Table Size: 60×40cm (24×15½")

Height Adjustment: 62-98cm

Recommended Max. Load: 10kg

Warranty Information

All Days overbed tables are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.



751



751C

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035

Fax: 03448 730 100

www.performancehealth.co.uk

International:

Tel: +44 1623 448 706

Fax: +44 1623 448 784

Performance Health Supply, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health Australia

Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

**PERFORMANCE
HEALTH®**