Rolyan® Preformed Functional Position Splints

Rolyan® Functional Position Hand Splint: A311-LS through A311-RL

Rolyan® Aquaplast® Functional Position Splint: A311-301 through A311-306


Rolyan® Functional Position Hand Splint with Strapping: A312-1 through A312-12 and A312-25 through A312-30

Rolyan® Perforated Functional Position Hand Splint with Strapping: A312-19 through A312-24

Rolyan® Functional Position Splint with Slot & Loop Strapping: A312-13 through A312-18

A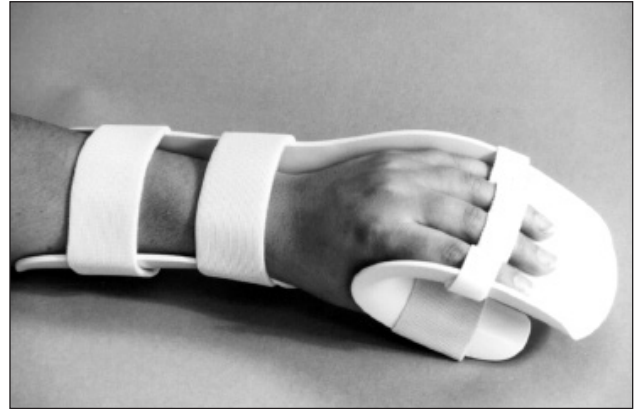

B

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed.

INDICATIONS

- To support inflamed joints or to immobilize joints after injury or surgery

CONTRAINDICATIONS

- Not for the acute phase of most wrist or hand fractures
- Not for use directly over open wounds and infected areas

PRECAUTIONS

- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

PROCEDURE

Fit

The splint can be modified for a better fit. Check the following areas for appropriate positioning and fit:

- The splint should cover two-thirds of the length of the forearm.
- The splint should clear bony prominences, and it should not impinge the thumb web space.
- The proximal edge of the splint can be flared for increased comfort.

Modification

Rolyan Functional Position Hand Splint

Rolyan Functional Position Hand Splint with Strapping (Product Nos. A312-1–6 and 25–30—see below for A312-7–12)

To trim or modify these splints, which are made from 1/8" (3.2mm) **Polyform®** splinting material, spot-heat the splint with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) for one minute or until pliable. **Heating the entire splint may cause it to lose its preformed shape.** Polyform has a high degree of drape—for better results when modifying the material, shape it with stroking movements.

Rolyan Perforated Functional Position Hand Splint with Strapping

This splint can be modified as described above. It should be trimmed above or below the perforations, not through them.

Rolyan Functional Position Splint with Slot & Loop Strapping

This splint can be modified as described above, but be careful not to heat the splint near the strapping or the plastic base for the slot-and-loop strapping.

Rolyan Functional Position Hand Splint with Strapping

 (Product Nos. A312-7–12)

Preformed from 1/16" (1.6mm) Polyform splinting material, the light splint can be trimmed without heating. If it requires reforming, the area needing modification should be spot-heated with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) for 30 seconds, or just until pliable. **A splint made from Light splinting material will lose its shape rapidly if submerged in hot water.**

Rolyan Aquaplast Functional Position Splint

To trim or modify these splints, which are made from 1/8" (3.2mm) Aquaplast splinting material, spot-heat the splint with a heat gun or in a heat pan of water at a temperature of 160° to 170°F (70°–75°C) for 2 minutes or until pliable. Smooth areas that have been trimmed with the palm of the hand.

Strapping

Apply straps as desired. Only some splints include strapping materials. Suggested strapping for most of the splints is shown in **A**. An additional strap may be placed across the wrist for increased support.

Rolyan Functional Position Splint with Slot & Loop Strapping

—Put the splint into position by sliding the fingers under the elastic strapping. If necessary, adjust the slot-and-loop strapping for proper fit—feed it through the slotted plastic base on the underside of the splint to increase or decrease the amount of strapping needed to secure each finger.

—Attach the two hook tabs to the ulnar and radial borders of the hand portion of the splint. Attach the elastic strap to the hook tabs.

—Attach forearm and thumb straps as shown in **B**. If desired, trim strap length. An additional strap may be placed across the wrist for increased support.

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Patterson Medical, Ltd.