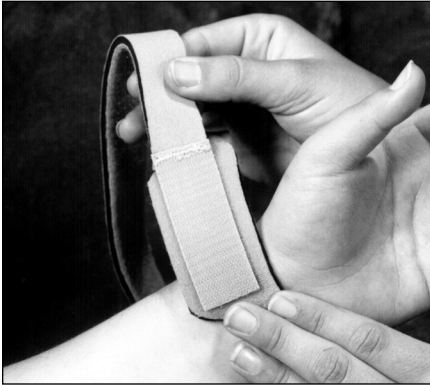
Rolyan® Thumb Loop:

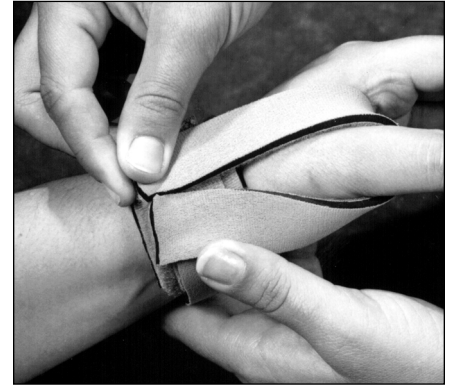
A834-90 through A834-94

A

B

C

INDICATIONS

- For positioning in the presence of mild to moderate flexor spasticity

CONTRAINDICATIONS

- Not for severe hypertonicity
- Not for use with severe edema
- Not for people who have an allergic reaction to neoprene or who have a tendency to develop dermatitis
- Not for use over open wounds and infected areas
- May not be for people with circulatory problems

PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**

- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the splint is not so tight that circulation is impaired.**

CARE AND CLEANING

- Fasten closures. Hand wash in warm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the splint is completely dry before reapplying it.
- Do not use ointments or oils under the splint.
- Do not dry clean or expose to heat or sunlight for extended periods of time.

APPLICATION

Cross Method—Provides radial pull of thumb to place into reflex-inhibiting position.

1. With wider base of pad distal and black side facing out, place CMC pad over radial aspect of wrist just proximal to CMC joint (A).
2. With black side toward patient's skin, place hook end of strap onto CMC pad (A).

3. Wrap strap around wrist and attach to hook end of strap. Then bring it around dorsum of thumb, through web space, and over thenar eminence. Cross over strap at base of thumb and attach to hook end of strap (B).

4. Trim strap length.

Alternate Method—Provides palmar positioning of thumb.

5. Follow instructions 1 and 2.
6. Wrap strap around wrist and attach to hook end of strap. Cut strap. Be sure that one strap piece will be long enough to wrap around wrist and the other around thumb.
7. Attach one end of second strap piece to hook on volar surface at base of thumb. Pull strap through web space and attach on dorsum of radial side of wrist (C).
8. Trim strap length.

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Patterson Medical Ltd.