Rolyan® Kwik-Form® Functional Resting Splint:

A812-700 through A812-703

R-Value® Kwik-Form® Functional Resting Splint:

A812-730 through A812-733

Rolyan® Kwik-Form® Progressive Hand Splint:

A812-620 through A812-621

R-Value® Kwik-Form® Progressive Hand Splint:

A812-630 through A812-631

Figures referred to in the instructions are on page 6.

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints.

INDICATIONS

- For mild to moderate hypertonicity

Kwik-Form Functional Resting Splint

- Can be used with arthritis

CONTRAINDICATIONS

- Not for severe hypertonicity

PRECAUTIONS

- These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the straps are not so tight that they interfere with circulation.
- Recommended for use by only one patient.

Kwik-Form Functional Resting Splint

- Do not attempt to modify a splint to fit the opposite hand.

CARE AND CLEANING

Rolyan Kwik-Form Splints

Rolyan Kwik-Form Splint bases have a removable cover. Before cleaning, remove the cover from the splint and close all fasteners.

- **Splint base**—Foam-covered splint base can be wiped clean with mild detergent. Allow to air dry thoroughly.

- **Fabric cover**—Place cover in mesh bag. Machine wash in warm water. Allow to air dry thoroughly before replacing on splint.

- **Replacing cover**—Replace cover on the Kwik-Form Functional Resting Splint as shown in **A** and on the Kwik-Form Progressive Hand Splint as shown in **B**. After replacing cover, be sure fabric is smooth on splint base.

R-Value Kwik-Form Splints

R-Value Kwik-Form Splints have hook-responsive material laminated to the splint surface. The splint base can be wiped clean with mild detergent. Allow to air dry thoroughly.

INSTRUCTIONS FOR USE

Rolyan Kwik-Form Functional Resting Splint

1. Put the splint into position on the extremity to determine necessary modification. Bend the splint base to accommodate the level of flexion or extension. Proper positioning is shown in **C** and **D**.
2. Apply the splint to the extremity and wrap the longer distal strap diagonally across the dorsum of the wrist, the volar surface of the forearm, and the dorsal surface of the forearm (**E**). Attach to the hook material on the proximal end of the cover. Strap length can be trimmed.
3. Wrap the shorter distal strap across the fingers.
4. Fasten the proximal strap.
5. A properly applied Rolyan Kwik-Form Functional Resting Splint is shown in **E**.

R-Value Kwik-Form Functional Resting Splint

The splint includes three straps with hook ends. The longest strap also has hook material attached to the center of one surface.

6. Put the splint into position on the extremity to determine necessary modification. Bend the splint base to accommodate the level of flexion or extension. Proper positioning is shown in **C** and **D**.

7. Apply the splint to the extremity. Place the hook piece in the center of the longest strap onto the volar surface of the splint at the level of the wrist. Wrap both strap ends diagonally across the dorsum of wrist and attach each to the opposite side of the splint (**F**).

8. Position one of the remaining straps distally and the other proximally.

9. A properly applied R-Value Kwik-Form Functional Resting Splint is shown in **F**.

Rolyan and R-Value Kwik-Form Progressive Hand Splints

The splint includes three straps with hook ends. The longest strap also has hook material attached to the center of one surface.

10. Put the splint into position on the extremity to determine necessary modification. Bend the splint base to accommodate the level of flexion or extension. The distal projection cannot be modified.

11. Attach the hook end of one strap to the ulnar surface of the distal volar projection as shown in **G**.

12. Choose the foam cone piece that best suits the level of flexion or extension. Put the cover on the cone and put the cone onto the distal projection of the splint over the hook end of the strap.

13. With the splint in position on the extremity, wrap the distal strap across the fingers. Fasten the strap to the hook on the cover of the cone, continue wrapping under the thumb, and fasten it to itself.

14. Place the hook piece in the center of the longest strap onto the volar surface of the splint at the level of the wrist. Wrap both strap ends diagonally across the dorsum of wrist and attach each to the opposite side of the splint (**H**).

15. Attach the remaining strap proximally.

16. A properly applied Kwik-Form Progressive Hand Splint is shown in **H**.

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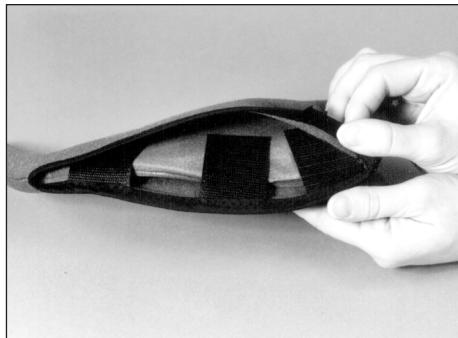
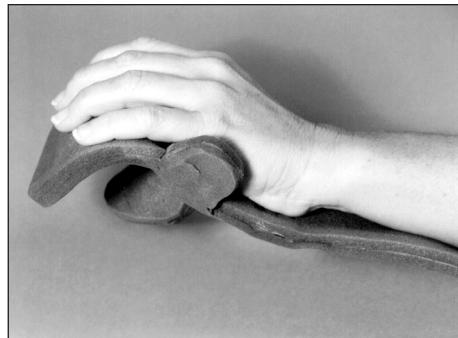
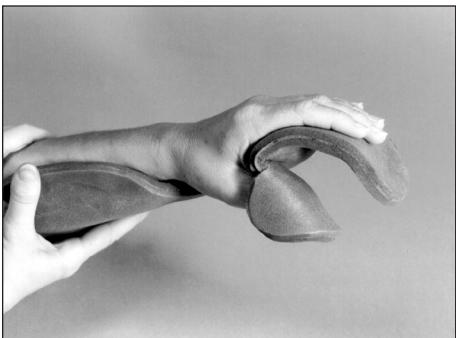
 **PATTERSON MEDICAL**
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012

 **PATTERSON MEDICAL**
Patterson Medical Canada
6675 Millcreek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Direct Phone: 905-858-6000
Direct Fax: 905-858-6001

 **homecraft**
a Patterson Brand

Authorized European Representative
Homecraft Rolyan
A Patterson Medical Company
Nunn Brook Road, Huthwaite
Sutton-In-Ashfield, Nottinghamshire NG17
2HU
United Kingdom
Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 1623 448 706
Fax: +44 1623 448 784