

Mini Massager - Instructions

The Mini Massager comes with one "C" battery and three rubber head attachments. Select the ball, spot or flat disc attachment most appropriate for the area being massaged. The Mini Massager cover may left on for large muscle areas.

Specifications

Power Source - DC 1.5V (1 x "C" cell)

C.P.M. - approx. 5,500/min

Amplitude - 0.06mm

Battery Life - Approximately 90 minutes of continual operation

Size - 42mm x 33mm x 135mm

Weight - 3.5 oz (100g) without battery

Instruction for Use

- 1. To remove battery, turn end cap unit anti-clockwise and pull.
- 2. Insert the size "C" battery, positive end first. (button end)
- 3. Remove the cover and snap the chosen attachment in place.

Instructions for Care

- 1. Keep out of direct sunlight or strong heat.
- Keep out of humid rooms like the bathroom.
- Remove battery when not in use.
- 4. Clean unit and attachments with mild soap or disinfectant.

Note:

- 1. Do not immerse the Mini Massager in water.
- 2. Do not use the Mini Massager in the throat or on the eye.
- 3. Check with a physician before using the Mini Massager on pregnant women.

 Patterson Medical

Patterson Medical
1000 Remington Blvd.
Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310