Rolyan® Foot Drop Splint:

A376-1 and A376-2

Rolyan® Adjustable Foot Drop Splint:

A676-A and A676-AX

Replacement Foam Inserts:

A378-1 and A378-2

Figures referred to in the instructions are on page 4.

Indications

- For plantar flexion contractures following tendon shortening or hypertonicity
- For helping prevent foot eversion and inversion

Contraindications

- Not for fixed bony contractures
- Not for use over open wounds and infected areas

Precautions

- **Single patient use only.**
- **These splints are not to be used during ambulation.**
- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, discomfort, or skin discoloration or breakdown occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**

Care and Cleaning

- Do not wash the foam insert. To increase comfort and maintain cleanliness of the insert, place a towel or sheet between the insert and the patient's skin.
- The splint can be cleaned with soap and lukewarm water. The straps can be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication.

Instructions for use

The splints include the following components:

- preformed splint of rigid high-temperature plastic
- high-temperature plastic triangular support base
- two-piece foam insert
- three foam straps
- two eyebolts

The replacement foam inserts include the following components:

- two-piece foam insert
- three foam straps
- two strips of double-faced tape

Attaching the triangular support base

1. Lay the splint so the posterior calf portion is on a flat surface.
2. With the long edge down, position the triangular support base against the external plantar surface of the splint (A). Lift the splint so the holes in the triangular base cover the protruding bolt sleeves.
3. Use the upper hole in the triangular base that provides the desired amount of rotation—the foot can be supported straight or with 15° of external or internal rotation (A).
4. Tighten the eyebolts into the bolt sleeves.
5. Fasten the nuts to the ends of the eyebolts. **The eyes of the bolts can be used to apply traction, but do not apply traction using an eyebolt that does not have the nut fastened.**

Adjusting the amount of flexion (Rolyan Adjustable Foot Drop Splint only)

Adjust plantar flexion using the holes in the metal side plates—they are marked in 5° increments (B).

6. Using a screwdriver, remove the proximal screw from both sides and loosen the distal screw.
7. While holding the proximal screw sleeve in place inside the splint, reposition the metal plate over the screw sleeve. Adjust both sides to the same amount of flexion.
8. Screw the proximal screws into the screw sleeves and tighten the distal screws.

Positioning the foam inserts

9. The smaller, flat foam piece goes into the foot portion of the splint, and the larger, curved piece goes into the calf portion with the cutout area distal.
10. Remove the protective covering from the double-faced tape on the splint and place foam pieces into the splint.
11. The foam can be modified slightly using an electric knife.

Attaching the straps

12. With the patient's extremity in the splint, attach one end of one of the foam straps to the hook on the plantar surface of the splint.
13. Wrap the strap around the top of the foot and attach it to the distal hook piece on the posterior of the calf.
14. Attach a second strap, using the same hook pieces, to cross over the first strap.
15. Wrap the remaining strap around the calf, attaching it to the proximal hook piece on the splint.

Replacing the foam inserts

16. Position the double-faced tape that comes with the foam inserts on the back of the foam pieces, remove protective covering, and place foam pieces in splint. Place the pieces as described in instruction 9.
17. A completed Rolyan Adjustable Foot Drop Splint is shown in C.

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Patterson Medical Ltd.