

538, 538D LIFT-UP SHOWER SEAT



538



538D

Introduction

The Days Healthcare 538 range of lift-up shower seats are made from powder coated steel tube with a base plate to secure it to the wall. All models in this range have the seat support rails hinged at the plate allowing it to be folded away and locked into an upright position. The 538D model has a height adjustable leg which is required when extra support is needed.

Directions for use

The mounting plate has mounting holes to secure it to the wall. It should be fitted by a competent person and secured to a wall capable of supporting the vertical load and leverage affected upon it by the user.

Leg adjustment (538D only)

To adjust the height, grasp the 'snap lock' between forefinger and thumb and pull it out. Slide the leg out to the required position and push the clip firmly back into the hole. **NOTE:** Set the leg and rail height before securing the mounting plate to the wall.



Snap Lock

Maintenance

The Days Healthcare 538 and 538D are designed for minimum maintenance and require the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasives or bleach based cleaners which may cause damage or discolouration. Periodically check the wall plate fittings and the pivot point of the rail are secure.

Specification

Model	Seat Size	Height Adjustment	Distance From Wall	Max. User Weight
538	49 x 28cm	N/A	38cm	114 Kg
538D	49 x 28 cm	38 - 53cm	38cm	160 Kg

Warranty Information

The Days Healthcare model no. 538 and 538D are warranted for one year from date of purchase due to faulty workmanship and materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non observance of the instructions set out above.

Days Healthcare is not responsible for any damage or injury due to improper or unsafe use of their Shower Stools.

