IT

Sostegno lombare rinforzato Vulkan Classic 3022

Ideale per

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**These user instructions are available in large print, please contact Performance Health Customer Services for a copy.
Please retain these instructions for future reference.**

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Manufactured for

Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk
International:
Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
USA
www.performancehealth.com

Performance Health ANZ
Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France
13 rue André Pingat
CS 10045
51724 Reims cedex
France
Tel: 00(33) 03 10 00 79 30
Fax: 00(33) 03 10 00 79 49

**PERFORMANCE
HEALTH®**

Vulkan Classic 3022 Reinforced Universal Back Support

Ideal for

- Suitable for anyone involved in activities that involve regular twisting, bending or lifting, central backache or pain radiating down as far as the buttocks
- Helps reduce unwanted twisting movements
- Flexible stays provide additional support and stability

Hazards & Cautions

Do not wear if warming or heat-generating liniments, ointments or creams have been applied.
Should an irritation or rash develop, discontinue use immediately and seek medical advice.
If your injury is severe or does not improve within three to four days seek medical advice.
Do not use in case of signs of wear and/or damage to the support
Ensure that the support is suitable for the user

Cleaning & Care

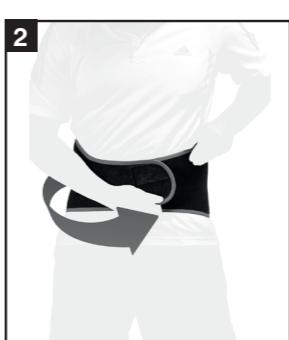
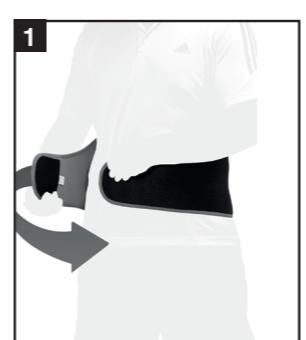
- Hand wash only below 40°C
- **Do Not** dry clean
- **Do Not** bleach
- **Do Not** iron
- **Do Not** tumble dry

Fitting Instructions

The support should be sized correctly to ensure maximum benefit. To select the correct size measure the circumference around your waist.

Ensure that the support is not uncomfortably tight and does not restrict blood flow.

Position the bottom of the support as low down on your back as possible, tighten until firm but comfortable and fasten at the front

Available sizes

09 156 7247

Fits all sizes

Material Composition

80% Neoprene, 8% Elastic, 7% Polyester, 3% Fibreglass, 2% Nylon.
Contains natural rubber.

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