IT

Ginocchiera a cerniera Vulkan Classic 3071

Ideale per

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These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for
Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:
Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:
Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
USA
www.performancehealth.com

Performance Health ANZ
Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France
13 rue André Pingat
CS 10045
51724 Reims cedex
France
Tel: 00(33) 03 10 00 79 30
Fax: 00(33) 03 10 00 79 49

**PERFORMANCE
HEALTH**

Vulkan Classic 3071 Hinged Knee Support

Ideal for

- Suitable for damaged medial or lateral ligaments, cartilage damage, rehabilitation following cruciate ligament injuries
- Dual pivot carbon fibre hinges control knee instability
- Adjustable tension straps provide additional compression and stability
- Open knee design with padded buttress helps support the patella

Hazards & Cautions

Do not wear if warming or heat-generating liniments, ointments or creams have been applied.

Should an irritation or rash develop, discontinue use immediately and seek medical advice.

If your injury is severe or does not improve within three to four days seek medical advice.

Do not use in case of signs of wear and/or damage to the support

Ensure that the support is suitable for the user

Cleaning & Care

- Hand wash only below 40°C
- Do Not dry clean
- Do Not bleach
- Do Not iron
- Do Not tumble dry

Fitting Instructions

The support should be sized correctly to ensure maximum benefit. To determine the correct size measure around the knee, over the kneecap with the leg held straight.

Ensure that the support is not uncomfortably tight and does not restrict blood flow.

Available sizes

Code	Size	Dimensions
09 156 7643	S	30 - 35cm / 11¾ - 13¾"
09 156 7650	M	35 - 40cm / 13¾ - 15¾"
09 156 7668	L	40 - 45cm / 15¾ - 17¾"
09 156 7676	XL	45 - 50cm / 17¾ - 19½"

Material Composition

70% Neoprene, 10% Elastic, 8% Nylon, 8% Carbon Fibre, 2% Polyester, 2% Polypropylene. Contains natural rubber.

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