


buckingham bra-angel user instructions


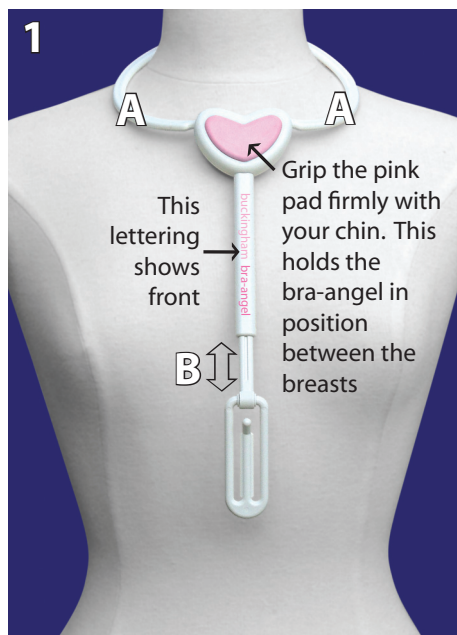


1
A A
Grip the pink pad firmly with your chin. This holds the bra-angel in position between the breasts
This lettering shows front
buckingham bra-angel
B


Sit down and position the bra-angel at your neck using the adjustable arms (A). Adjust its length (B) so the gripper at the bottom is below the breasts.




2
Place the bra upside down, inside out and back-to-front around your waist.

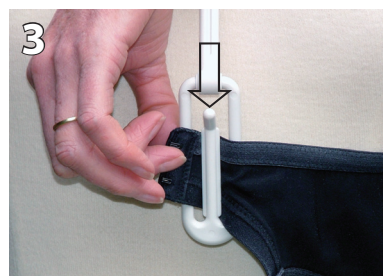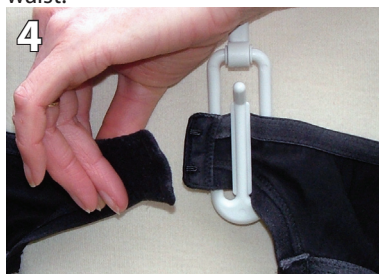

3
Feed the 'hook' end into the gripper.

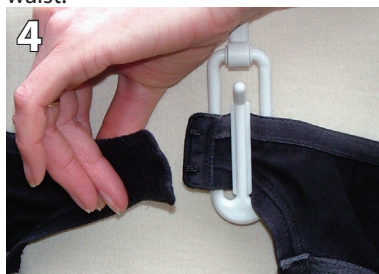

4
With the 'hook' end held firmly in the gripper reach for the 'eye' end and join the two ends together.

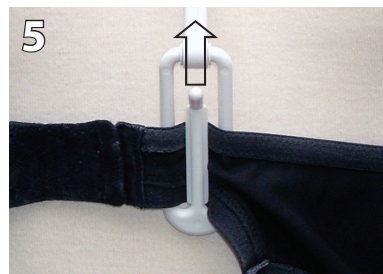

5
The bra can now be removed from the gripper and moved around the body. Remove the bra-angel to complete dressing.

Care Instructions

Wipe clean with a damp cloth. Do not immerse in water or use harsh abrasives.

Caution

This product is only to be used for the intended purpose of a dressing aid.

Keep out of reach of babies and young children. It should not be used as a play toy.

Latex free

Manufactured in China for Buckingham Healthcare Limited, Portsmouth PO3 6BQ, England
www.buckinghamhealthcare.co.uk

International patents pending.

