## Description

This brace helps to support mild to moderate ankle sprains and chronic ankle instabilities.

The rigid contoured outer shells control inversion or eversion whilst allowing normal dorsi-flexion. Foam padding evenly distributes pressure along with anatomically shaped plastic shells to provide protection.

This support is manufactured from high quality materials and we recommend that a medical professional fits the support for you. If at any time you experience discomfort or your condition worsens, remove the support immediately and consult your therapist/medical professional.

## Fitting Instructions

- A) Place the heel strap underneath the heel with sides of the brace on each side of the ankle
- B) Adjust the heel strap so the gel covering on each side of the ankle bone is firmly fitted
- C) Pull both of the leg straps and attach so the brace remains firmly in position

## Cleaning and Care

Hand wash in warm water with liquid mild soap and leave to dry naturally.

**DO NOT MACHINE WASH OR TUMBLE DRY  
NEVER OVER TIGHTEN STRAPS.**

This support should feel comfortable at all times.  
**REMOVE BEFORE GOING TO SLEEP.**

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### Performance Health Supply, Inc.