Shoulder Immobiliser/Abduction

Description

This support immobilises the shoulder after rotator cuff repair, shoulder girdle fracture, dislocation/subluxation or multidirectional instability. Made from a soft velour material for comfort and has easy use features such as a quick release buckle, de-rotational shoulder strap and an exercise ball to facilitate hand function and stimulates circulation.

This support is manufactured from high quality materials and we recommend that a medical professional fits the support for you. If at any time you experience discomfort or your condition worsens, remove the support immediately and consult your therapist/medical professional. For size measure the length of the forearm.

Code	Size	Cm
09 136 9131	Small	25-28
09 136 9123	Medium	28-30
09 136 9115	Large	30-33

Fitting Instructions

- Detach shoulder strap and open front panel. Position elbow in the sling as far back as possible. Loop the sling strap through the D-ring and secure.
- Secure strap at the top of the sling. If the thumb strap is desired, attach strap at the front of the sling.
- Place shoulder strap over the opposite shoulder. Connect shoulder strap to the sling using the quick release buckle.
- Place pillow at the waistline of the affected side. Attach the sling to the outside of the pillow, along the hook and loop strips. Buckle the waist strap to the pillow and adjust the strap to fit. Position the arm for desired internal or external rotation by sliding the pillow forward or back along the waistline.

Cleaning and Care

Hand wash in warm water with liquid mild soap and leave to dry naturally.

DO NOT MACHINE WASH OR TUMBLE DRY.

NEVER OVER TIGHTEN STRAPS.

This support should feel comfortable at all times.

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Performance Health International Ltd.
Performance Health Supply, Inc.