Anuform[®]

Instruction Manual – GB

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Manual de Instrucciones – ES

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Manuale d'Istruzioni – IT

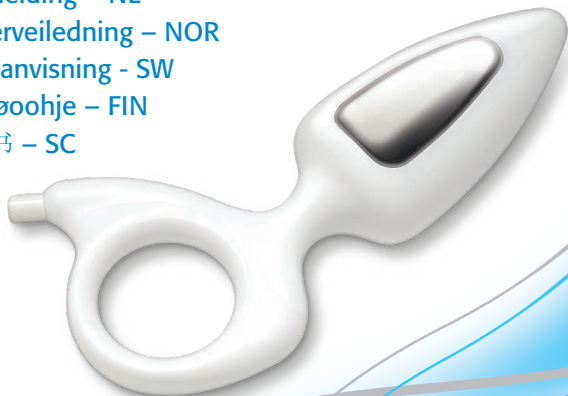
Handleiding – NL

Brukerveiledning – NOR

Bruksanvisning - SW

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inner strength, outer confidence

Specifications

Materials:

Probe body: High Impact Polystyrene

Electrodes: Medical Grade Stainless Steel

Cables: PVC over copper tinsel conductors

Weight: 23.5g, Cable Length: 30cm

Connectors: 2 x 2mm diameter sockets

Electrode Surface Area: 3cm² per electrode

'In Use' product lifetime: 6 months

Contents

- 2 - 3** Instructions for Use - GB
- 4 - 5** Conseils d'utilisation - FR
- 6 - 7** Gebrauchsanweisung - DE
- 8 - 9** Instrucciones de Uso - ES
- 10 - 11** Instruções de utilização - PORT
- 12 - 13** Istruzioni d'uso - IT
- 14 - 15** Gebruiksaanwijzing - NL
- 16 - 17** Brukerveiledning - NOR
- 18 - 19** Bruksanvisning - SW
- 20 - 21** Käyttöohjeet - FIN
- 22 - 23** 使用说明 - SC
- 24** Specifications

Indications for Use:

For use as an intra-anal probe in males and females and a small vaginal probe in females. For use only with muscle stimulation and EMG biofeedback equipment in the treatment of 'Stress', 'Urge' and 'Mixed' incontinence and faecal incontinence. It is recommended that this device be used only with Neen Peritone or Neen Pericalm units.

FOR SINGLE PATIENT USE – may be re-used by the same patient.

FOR SINGLE ORIFICE USE – not for both vaginal and anal use.

Use as directed by your medical adviser.

Contraindications & Precautions:**DO NOT USE:**

With electrical stimulation if fitted with a cardiac pacemaker • If pregnant • If the package seal is not intact • If an infection or tissue damage or recent surgical scar is present • Vaginally during menstruation • With electrical stimulation in patients with a history of carcinoma at the site of stimulation • If an irritation or discomfort occurs and seek medical guidance.

CAUTION IS ADVISED:

When using electrical stimulation on patients with altered or reduced tissue sensation • During electrical stimulation if the user has a history of epilepsy - seek informed medical guidance • When using the ANUFORM® with equipment capable of delivering outputs in excess of 2mA/cm². If in doubt contact the equipment manufacturer or your medical adviser.

Cleaning:

- Wash ANUFORM® in warm soapy water.
- Rinse in clean water and dry.
- Repeat after each use and dry thoroughly before storing in its plastic bag.
- DO NOT use boiling water.
- Avoid immersion of the lead wires in water when cleaning
- Do not autoclave

Disposal:

Clean thoroughly and dispose of according to local regulations.

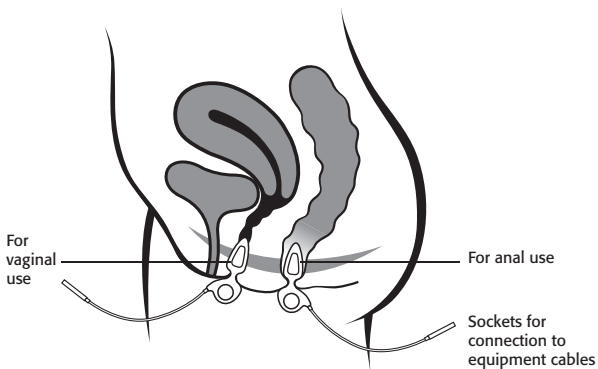
Contains nickel:

May cause irritation or allergic reaction following prolonged contact.

Always follow the user instructions.

Indications for Use:**IMPORTANT:**

- Before first and subsequent use wash and dry as directed.
- Apply light coating of KY Jelly or similar lubricating gel to tip and metal surfaces of probe to aid insertion and provide good electrode conductivity.
- **Anal use:** Insert into anus while 'bearing down' as if passing stool. 'Ring' part of ANUFORM® should remain external to the anus at all times.
- **Vaginal Use:** Insert into vagina in an 'East/West' position, i.e. with the metal electrode surfaces facing towards the hips. Insert until body of ANUFORM® is within the vaginal vault. 'Ring' part of probe should remain externally within labia.
- Ensure equipment is turned off before connecting to probe cables.
- Use as directed by your medical adviser. After use, turn off equipment, disconnect cables. Do not pull on cables to remove probe.
- Wash, dry and store as instructed on previous page.

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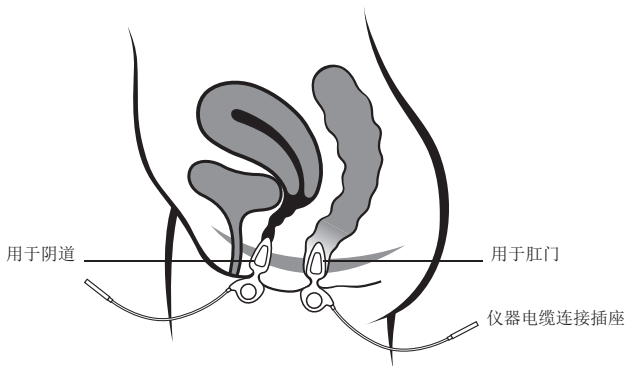
For more information on this and other products in the NEEN® range please visit:
www.neenpelvichealth.com

inner strength, outer confidence

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for:

 Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:
Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk
International:
Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.
W68 N158 Evergreen Blvd,
Cedarburg, WI 53012
USA
www.performancehealth.com

Performance Health ANZ
Unit 3, 3 Basalt Road,
Penmullwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France
13 rue André Pingat,
CS 10045
51724 Reims Cedex
France
Tel: 00(33) 03 10 00 79 30
Fax: 00(33) 03 10 00 79 49

**PERFORMANCE
HEALTH**