ENGLISH

Please keep these instructions for future reference.

Intended Uses :

- ▶ Bath steps provide a platform on the outside of the bath, lessening the height over which people need to lift their legs to get into and out of the bath.
- ▶ A bathstep may not be suitable for anyone whose balance is suspect. This slip-resistant step provides a safer surface to stand on. A wall mounted grab rail will provide a secure handhold.
- ▶ The Prima Bath Step is of a modular design. The top section may be used on its own or with one or both of the supplied raising sections. Up to a maximum of 10 additional sections may be used. More than one bath step may be joined at the same, or different, heights by securely pushing raising sections in to the top section, and each other using the slots underneath and the jointing pieces provided. **Before use, 4 rubber feet must be fitted to the section in contact with the floor.**

1. Ensure the Prima Bath Step is dry and free from soap or oil film.
2. Position the step up to the side of bath.
3. To adjust the height of the step, add or remove raising sections ensuring when at the desired height, they are firmly pushed together.
4. Stand centrally on the step with both feet.
5. Test for stability.
6. Step directly into the bath.
7. The product is designed for use in a static mode only. As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement during use.
8. Clean with non-scratch detergent.

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