

User Instructions for Blanket Cradles

B48 – B49 – B490


Introduction

Congratulations on your purchase of the Cefndy Healthcare Blanket Cradle. Please ensure you read and understand these instructions in full before construction and use. Please retain these for future reference.

The blanket cradle is designed to lift blankets off of the legs. The product frame is static during use and not fixed. The cradle fits under the mattress and bed clothes.

There are three versions of blanket cradle available. The B48 Folding Blanket Cradle, the B49 Rigid Blanket Cradle and the B490 Adjustable Blanket Cradle.

Performance Health #	081566876	081305671	
Specifications			
	B48	B49	B490
Folded Length	510mm	---	---
Folded Width	362mm	---	---
Folded Depth	63mm	---	---
Upright Height	495mm	---	---
Width of Frame	362mm	413mm	---
Length of Frame	510mm	610mm	472mm
Height	495mm	457mm	450 – 700mm

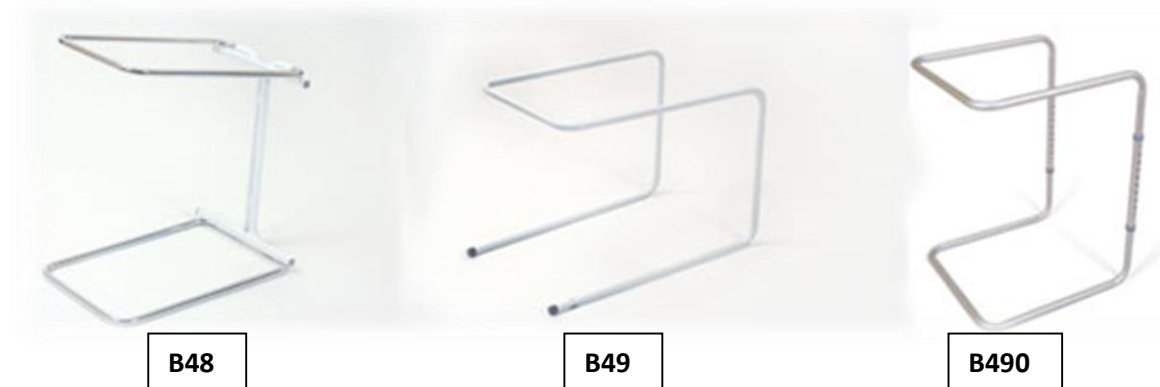

Pack Contents

- Blanket Cradle
- User Instructions

Assembly Instructions

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**
- Slide the end of the cradle under the mattress to the desired position.
Blankets can now be draped over the cradle.

Correct Use

- The product is designed to help the user by lifting bed blankets off their legs.
- The cradle is not intended to support any heavy weight. Please refrain from using it to support any body weight.
- The cradle is designed to be used in conjunction with a bed.
- Do not attempt to overload the cradle.

Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty.

Use of power/jet washing & industrial washing machines

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that rails are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the inside of the rail as it could cause permanent rusting of the rail (leave rails in a warm environment for several hours to ensure all moisture evaporates).
- We cannot offer advice on industrial washing machine with regards to specific settings as these will be dependent on the machines manufacturer's instructions.

Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should

damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Warning

- This product is intended for light use i.e. blankets, duvets etc. – It is not designed to support heavy loads.
- We recommend a full risk assessment should be carried out before use.

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of in line with local recycling policies.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG

Tel: 01824 706005

E-mail: cefndy.sales@denbighshire.gov.uk