

homecraft Aluminium Shower Commode Chair

CODE	DESCRIPTION
091438738	Shower Commode Chair
091438431	Optional footrest
091081280	Spare Square Bowl
091469287	Spare White Aperture Seat

Introduction

The Homecraft Aluminium Shower Commode Chair is manufactured from anodised aluminium tubing with a commode aperture which allows the chairs to be used as a commode or over a toilet pedestal. The back support incorporates a pushing handle for attendant use. Hook-on footrests are optional extra.

Assembly

1. Remove the nuts, bolts and saddle spacers from the back rest assembly.
2. Assemble one side of the backrest to the frame using two bolts, saddle spacers and nuts, ensuring two saddle spacers go back to back between the tubes.
3. Pull the other side of the backrest frame to align the holes and fit the bolts (this operation may require some stretching of the backrest upholstery).
4. Fit the remaining saddle spacers and nuts and tighten the bolts to secure the backrest.

Operating Instructions

The Homecraft Aluminium Shower Commode Chair is attendant controlled. To apply the brakes depress the lever on the rear of the castors. This prevents the wheels from rotating or swivelling. Release the brakes by pulling up the lever. The footrests hook onto the two spacers between the front legs and the cross brace.

The footplates pivot up to allow entry.

The commode bowl slides onto the runners under the seat and is fitted and removed from the back.

Cautions

- Ensure the shower commode chair is suitable for the intended user.
- Do not exceed the maximum user weight.
- Do not stand on the footrests when getting into or out of the chair as this will tend to tip it forwards.
- Always apply brakes when getting in or out of the shower chair.
- Remember in a shower environment the floors are wet and slippery. Be aware that if the user leans far forward in the chair and takes the weight off the rear wheels the shower chair will tend to slide backwards even with the brakes applied.
- Do not sit on the front edge of the seat as this will also tend to tip the chair forward.
- Maintain an upright posture and lean against the backrest.
- Ensure that the chair is checked before each use for signs of damage or wear.
- Ensure all parts are secure before each use.

WARNING! Always take extra care when getting into or out of the Shower Commode chair. Supervision is essential while getting on or off the Shower Commode chair.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for
Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:
Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:
Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
USA
www.performancehealth.com

Performance Health ANZ
Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France
13 rue André Pingat
CS 10045
51724 Reims cedex
France
Tel: 00(33) 03 10 00 79 30
Fax: 00(33) 03 10 00 79 49

**PERFORMANCE
HEALTH**

Maintenance

The Homecraft Aluminium Shower Commode Chair is designed for minimum maintenance and easy, modular replacement of parts for long life. To clean, wipe over with a soft cloth. Do not use harsh abrasives. Use general purpose neutral detergent and warm water. The shower chair may be placed under running water at low pressure or showered down to assist cleaning.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry. Daily, before use, check the effective operation of the brakes and castors.

Specification

Width	Depth	Height	Seat Width	Seat Depth	Seat Height	Max. User Weight
530mm	610mm	1040mm	430mm	430mm	560mm	100kg

Warranty Information

Performance Health warrants that the shower commode chair is free from defects in material and workmanship. The warranty shall remain in effect for 1 year from the date of original consumer purchase.

The warranty does not extend to parts that are exposed to natural wear and tear that will require periodic replacement.

This warranty does not cover the following:

- Replacement parts supplied by anyone other than Performance Health or the stockist;
- Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use;
- Performance Health is not liable for incidental or consequential injury or damage to property;

Should it be necessary to return the shower commode chair, it is essential that it is adequately packed to protect it in transit, preferably in its original packaging, otherwise Performance Health cannot accept any responsibility for transit damage to the unit.

This does not affect your statutory rights.
