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homecraft Adjustable Height Comfort Commode

	New Code	Old Code
Homecraft Adjustable Height Comfort Commode	09 130 9798	521A
Homecraft Adjustable Height Comfort Commode With Removable Armrests	09 132 5828	521AD
Homecraft Adjustable Height Comfort Commode With Padded Toilet Seat	09 143 8993	11240

Intended uses

Commodes allow for, or increase, independence when requiring the toilet, particularly by compensating for impaired movement or disability.

Description

Modern, attractive commode that offers comfort and stability within any home or institutional environment. The padded blue seat cover and clip-on black toilet seat are easily removed for cleaning. The deluxe model has a padded blue toilet seat that provides the option of additional comfort. The commode pan with handle is simple to remove for waste disposal. The height adjustment allows the commode to be tailored to the required position for the user. Available with or without removable armrests.

Instructions for use

1. Before use make sure the commode is stable on a flat horizontal surface.
2. To adjust the height of the commode, remove the pin-clips and slide the leg extension to the desired height. Be sure all of the legs are adjusted to the same height and replace the pin-clips.
3. Insert the commode pan. Ensure the handle position of the commode pan is resting on the front bar of the commode. Please ensure the bucket is placed underneath whilst in the seating position throughout the duration.
4. Clip the black toilet seat onto the rear cross bar, ensuring the seat clips into position. For the deluxe model, clip the blue padded toilet seat onto the front and rear cross bars.
5. On model 521AD (09 132 5828) the armrests can be removed by lifting them out of the sockets, enabling side transfer to/from the commode. Ensure the armrests are replaced and fitted securely before use.

Care and cleaning

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasives. To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectants (haztabs/precept), rinse and dry.

⚠ Cautions

- These commodes are not to be used to support more than 190kg/419lbs/30st
- Before use make sure the commode is stable on a flat horizontal surface
- Under no circumstances should the commode be moved whilst occupied
- Do not drag the commodes across the floor
- Before each use, ensure the pin-clips, seat and armrests are fitted correctly
- Regular checks of the ferrules should be made, ensuring that the commode is suitable for use

Maximum user weight

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for
Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.
UK:
Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk
International:
Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.
28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938, USA.
Tel: 1-800-323-5547
Telefax: 1-800-547-4333

Performance Health ANZ
Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France
Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15
Fax: 03 24 52 90 34

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