

Introduction

The Days® 753C is an adjustable height overbed table made from chromium plated and powder coated steel tubing and can also be used at a chair or wheelchair. The easy-clean laminated teak top has raised edges with open corners and can be tilted for reading.

The Days® 753C model is fitted with castors, two of which are braked to avoid movement of the table when it is in the desired position.

Assembly

Fit the U bend (Fig 2 'A') on to the base section (Fig 2 'B'). Tighten up with the hand knobs provided. (Fig 2 'C').

Place the chrome legs (Fig 3 'A') into the bottom section (Fig 3 'B') and tighten up with the hand knobs provided.

Height Adjustment

The table height can be adjusted by slackening the two black hand knobs and adjusting the table top to the desired position then tightening the hand knobs (Fig 3).

Before use make sure that both hand knobs are securely tightened.

Maintenance

The Days® 753C overbed table is designed for minimum maintenance.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Warranty Information

The Days® overbed table is warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.


The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

Fig 1



Fig 2

Fig 3

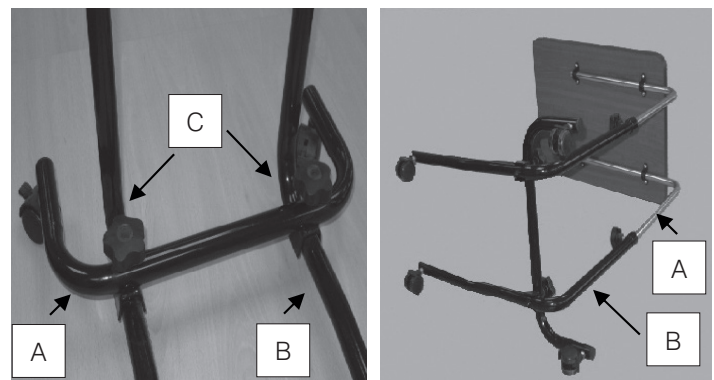

Specifications

Table Size:	54 x 40 cm
Height Adjustment:	53 - 82 cm
Recommended Max. Load:	10 kg

CAUTION! - The overbed tables are not designed to support heavy weight. Users must not lean on them or use them for support when getting up or sitting down.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
USA
www.performancehealth.com

Performance Health ANZ

Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

13 rue André Pingat
CS 10045
51724 Reims cedex
France
Tel: 00(33) 03 10 00 79 30
Fax: 00(33) 03 10 00 79 49