NEW CODE	OLD CODE
09 112 0633	AA8810
09 112 0724	AA8814

Intended Uses

To compensate for a disability by helping a person transfer from one sitting surface to another.

Instructions For Use

With the flat side down, position one end of the board under the user's body and the other end on the destination surface. Position the board so it supports the full body weight during transfer. The smooth surface of the board is best suited for clothing transfers. For bare skin transfers, a towel can be placed over the board for a smoother transfer.

⚠ Cautions

- It is recommended that the Wooden Transfer Board are not used without previous training by a healthcare professional. For some people, attempting a transfer without assistance from a caregiver is not recommended.
- The board is not to be used to support more than 100kg / 15st.
- Inspect the board prior to each use for signs of wear. Do not use the board if it appears weakened.
- Be sure the board is securely in place before putting weight on it.
- If unusual swelling, skin discolouration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for
Performance Health International Ltd.
 Nunn Brook Road, Huthwaite,
 Sutton-in-Ashfield, Nottinghamshire,
 NG17 2HU, UK.

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Performance Health Supply, Inc.